

## TO GO FURTHER

The preparation to be injected may be **contaminated** by bacteria and fungi coming from various sources:

- soiled hands,
- preparation of a drug itself contaminated,
- non sterile preparation tools or water.

➔ **Filtering with Sterifilt® eliminates bacteria and fungi from your mix.**

Whether it is powder or crushed medicine, **the product you inject contains insoluble particles.**

They damage your veins and may lead to serious medical complications in the long run.

➔ **Filtering with Sterifilt® eliminates insoluble particles from your mix, while preserving the effect of the active substance.**

This medical device is a regulated health product that bears, under this regulation, the CE marking.  
Carefully read the instructions on the label.  
November 2021 release

## REDUCE HARMS!

### INFECTIONS

Most infections associated with injection are caused by bacteria and fungi present on the skin and in the mouth.

#### To prevent them:

- ➔ Wash hands with soap and water or rub them with an alcohol-based solution
- ➔ Use sterile equipment to prepare the mix as well as to inject it
- ➔ Filter your preparation with Sterifilt®
- ➔ Disinfect the injection site with an alcohol-based swab
- ➔ Do not lick the needle!

### VIRAL TRANSMISSIONS (HIV, HEPATITIS)

The only way to prevent them is to **never share your preparation nor injection equipment.**



Scan this QR code to find out our videos and other resources

Apothicom Distribution - 52 avenue Edison - 75013 Paris (France)  
Tél : +33 (0)1 53 61 18 41 - [infos@apothicom.org](mailto:infos@apothicom.org)  
[www.apothicom.org](http://www.apothicom.org)



### THE HIGHEST LEVEL OF FILTRATION!

- Avoid “cotton fever”, abscesses and other infections
- Protect your veins...

... By eliminating **bacteria, fungi** and **particles** in one single step.

  
**Apothicom**  
[www.apothicom.org](http://www.apothicom.org)

## WHY?

Sterifilt® offers **the highest level of protection by filtration:**

it eliminates insoluble particles, as well as bacteria and fungi.

**You thus protect** your veins and prevent abscesses, “cotton fever”, “puffy hands” and many other non viral infections.

### The Sterifilt® assets



- Fits on most syringes.
- Easy to use in a one and single step.
- Quickly filters powders and tablets/capsules.
- Retains less product.
- Does not remove the active product, nor impair its effect.

## HOW?



**Open the Sterifilt® package.**

Tilt the filter, **without removing it from the package** and making sure **not to touch it directly with your fingers.**

1

2

**Insert the syringe tip into the Sterifilt®.**

If you use a needleless syringe, place the filter where the needle goes.

Once the filter is in place, firmly press and twist the syringe in the filter to **ensure a good seal.**



Hold the syringe and the filter **without flattening the filter grid against the bottom of the cooker** to avoid a suction effect.

**Pull the plunger back slowly.**

Remove and discard the Sterifilt®.

3

## TIPS

- Prior to first use, practice in a calm environment using water.
- You are having trouble drawing the liquid into the syringe?
  - Make sure the seal between the Sterifilt® and the syringe is adequately tight (step 2).
  - The filter touches the bottom of the cooker: avoid suction effect by slightly inclining the syringe (step 3).