

# Stericup®

A sterile, single use cooker



In a range of colours to reduce accidental sharing

**Packaging:** Individual sterile blister pack.

**Composition:** 2,5 ml aluminium sterile spoon in a range of colours.

**Objectives:** To reduce harm arising from sharing and re-use of mixing spoons/cookers, such as the transmission of hepatitis B and C and HIV, abscesses, endocarditis and cellulitis from bacterial contamination. To encourage the use of aseptic mixing and safer injecting techniques.

**Indications:** The prevention of infections related to the sharing of spoons/cookers. Helps to reduce the risk of infections arising from sharing, borrowing and re-use of mixing spoons/cookers. The availability of five different colours reinforces the importance of using own equipment.

**Caution:** Fingers should never touch the inside of the Stericup®—any bacteria, fungi or viruses such as the Hepatitis C Virus (ie. in traces of blood) present on the surface of the skin may contaminate the drug-mix being prepared for injection. Never share or draw-up after someone else has used the spoon. Even if it has been cleaned, the spoon can still contain viruses from someone else's syringe, filter, water or fingers. One jab is all it takes to get infected ! Never re-use someone else's works.

**History:** Developed in France in 1996 with input from needle exchange programs.

Safety and efficacy validated through exhaustive laboratory testing using brown heroin (base), white heroin (hydrochloride), cocaine (hydrochloride), crack (cocaine base), amphetamines and crushed tablets. Approved as a therapeutic device (CE mark) and distributed throughout France with government support to all pharmacies and needle and syringe programs.

Distributed and used worldwide by several Public Health Services.

**Directions for use:** Press down on the Stericup® handle to remove it from the blister. Take care to avoid touching the inside of the cup. Squeeze the plastic handle between thumb and index finger, and insert the Stericup®'s aluminium handle. The covered Stericup® handle protects the user's fingers from hot metal during the "cooking" process.

To avoid any accidental sharing of "cookers", users are advised to take note of and remember the colour of the particular Stericup they have chosen.

Stericup® is a CE registered medical device.



[www.apothicom.org](http://www.apothicom.org)

APOTHICOM Distribution, 52 avenue Edison, 75013. FRANCE

Phone: + 33 (0)1 53 61 18 41. Fax: + 33 (0)1 53 61 04 49 . E-mail: [infos@apothicom.org](mailto:infos@apothicom.org)

# Stericup

## Order Form

Complete this form and post to:

APOTHICOM Distribution/52 avenue Edison. 75013 Paris. FRANCE  
Tél.: 00 33 (0)1 53 61 18 41. Fax: 00 33 (0)1 53 61 04 49. E-mail: [infos@apothicom.org](mailto:infos@apothicom.org)

Please fill in using capital letters

NAME: ..... FIRST NAME: .....

JOB TITLE: ..... ORGANISATION: .....

DELIVERY ADDRESS: .....

INVOICE ADDRESS (if different): .....

PHONE NUMBER: ..... FAX NUMBER: .....

E-MAIL ADDRESS: .....@ .....

To make ordering simpler, Stericup® is available and priced, in multiples of 1,000. We aim to deliver all orders promptly; if you require an urgent delivery please call. Please fax your order to: +33 1 53 61 04 49



Stericup® (with cotton filter + dry post injection swab + handle)				Quantity	Total
From 1000 to 5000	Per thousand	Ex Vat 165,00	Vat incl. 197,34		
6000 and more	Per thousand	Ex Vat 160,00	Vat incl. 191,36		
Shipping costs are not included					
Sterimix® (Stericup® + Sterifilt® + dry post injection swab + handle)				Quantity	Total
From 1000 to 5000	Per thousand	Ex Vat 195,00	Vat incl. 233,22		
6000 and more	Per thousand	Ex Vat 190,00	Vat incl. 227,24		
Sterifilt Leaflets	10 per 1000	Free			Free
Sterifilt Poster	1 per 1000	Free			Free
Sterifilt information cards	20 per 1000	Free			Free
Shipping costs are not included					

All items are priced ex shipping. Transport costs can be quoted according to quantities, please let us know if you would need a prior quotation. For the European Union countries only: if you are VAT registered please specify your intracommunautaire number on the above form, so that we shall not add French VAT rate (19,6%) to the invoice amount.

Payment Method: by transfer bank or by cheque to Apothicom.

Date:

Signature:

Update notice: May 2009. Please cancel all previous and invalid notices

For further information • e-mail: [infos@apothicom.org](mailto:infos@apothicom.org) or call + 33 (0)1 53 61 18 41

**Bank**

Name: Apothicom Distribution

IBAN Number: FR76 3000 4003 1700 0100 6193 565

BIC: BNPAFRPPBY